



LIFETIME REPLACEMENT WARRANTY

If you want to replace your WalkFit® Platinum Orthotics for any reason, just return them to: 5360 Capital Ct. Ste. 103 Reno, NV 89502. Please include your payment of \$9.95 shipping & processing (per pair) made payable to Ideal Living, LLC. Please write WalkFit® in the memo section of your payment. We will exchange them for FREE, FOREVER.

1 INFO

NAME _____

ADDRESS _____

CITY _____

STATE _____

ZIP _____

2 QUANTITY



SIZE	QUANTITY
SIZE	QUANTITY
SIZE	QUANTITY



SIZE	QUANTITY
SIZE	QUANTITY
SIZE	QUANTITY

SIZE	MEN	WOMEN
A		5-5.5
B		6-6.5
C	6-6.5	7-7.5
D	7-7.5	8-8.5
E	8-8.5	9-9.5
F	9-9.5	10-10.5
G	10-10.5	11-11.5
H	11-11.5	12-12.5
I	12-12.5	
J	13-13.5	
K	14-14.5	

3 PAYMENT

Check payable to Ideal Living, LLC enclosed (include \$9.95 shipping & processing per pair)

Credit Card Orders: Please call Customer Service at 877.264.9243 Hours: M-F / 7am-4pm PT and Sat / 7am-3pm PT

SANDAL CONVERSION KIT INSTRUCTIONS

Enjoy this sandal conversion kit as our free gift to you for purchasing WalkFit® Platinum orthotics.


Now you can get the benefits of our orthotics in your favorite pair of sandals. The WalkFit® Platinum orthotics sandal conversion kit contains eight white circular pairs of hook and loop adhesives. Some of the hook and loop adhesive circles feel soft to the touch and some feel rough and prickly like sandpaper. After you have located all the hook and loop adhesive circle pairs and separated the "rough" ones from the "soft" ones, proceed as follows:

- 1 Peel the "rough" hook-side adhesive circles off of the strip that they came on, and attach adhesive-side down to the bottoms of your orthotics – two near the front portion, and two near the back (as shown in the diagram below). You can always pull them off and re-adjust their positioning to find what works best for you.
- 2 Next, place the "soft" loop-side of the adhesive circles inside your sandals, coordinating their placement to line up with the "rough" circles attached to the bottoms of the WalkFit® Platinum orthotic. Make sure that all hook and loop circles are lined up and firmly attached before use to ensure your safety, comfort and optimal WalkFit® Platinum orthotic performance.

*When your hook and loop adhesives start to wear off, replace immediately with new adhesive circles (usually available at craft and/or sewing stores nationwide).

Important
READ ALL INSTRUCTION MATERIALS BEFORE USE OF THE WALKFIT® PLATINUM ORTHOTICS AND THIS SANDAL CONVERSION KIT. IF YOU HAVE ANY CONCERNS ABOUT WHETHER OR NOT YOU SHOULD USE YOUR ORTHOTICS WITH SANDALS, PLEASE CONSULT WITH YOUR PHYSICIAN.

WFP-0622_R7

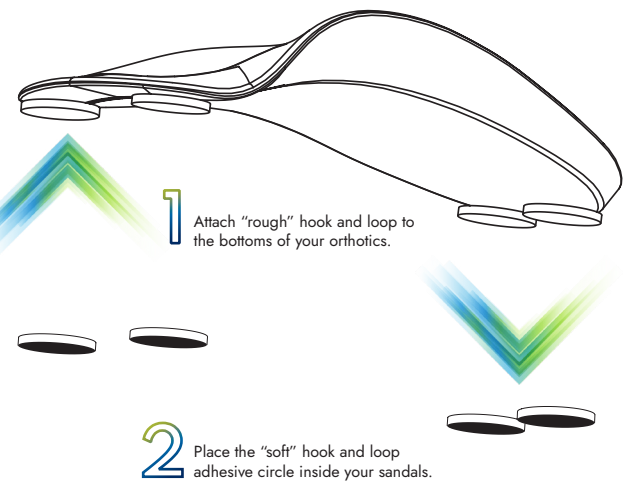

Walkfit® PLATINUM



INSTRUCTIONS
www.walkfitplatinum.com

IMPORTANT

DO NOT put WalkFit® Platinum orthotics into your shoes until you have read **ALL** the enclosed material. Please see special warning at the bottom of this page.

Congratulations on your purchase of WalkFit® Platinum orthotics. We know how wonderful they are and how much they will improve your comfort when standing, walking or running. However, before you wear them for the first time, it is important to read the following instructions.

SAFETY INFORMATION

People with diabetes or circulation problems should consult their podiatrist or health care professional before using this product to ensure appropriate treatment and care.

INSTRUCTIONS

1

Remove Extra Padding From Shoes

Some shoes and sneakers come with built-in arch supports and extra padding. WalkFit® Platinum orthotics replace these built-in arch supports. To receive the proper result from WalkFit® Platinum orthotics, remove the built-in supports and all extra padding. Leaving in arch supports and padding may cause significant discomfort when wearing those shoes.

2

Customizable Inserts

WalkFit® Platinum orthotics come with three arch inserts: LOW, MED, and HIGH. The height of the inserts are written on the back of each. The LOW inserts are already inserted for you. To switch to another pair of inserts, simply bend back the top of your orthotics and lift out the inserts. Place the new inserts in position and press down all over to ensure that they are securely in place. Try out each level of inserts to see which one feels most comfortable initially. If MED or HIGH seems most comfortable, choose MED or HIGH. For better comfort, WalkFit® Platinum orthotics come with special shock-absorbing heel support. Please note this special shock-absorbing heel support is not interchangeable.

New Massaging Arch Insert

This insert features hundreds of nodules contoured to your foot for added comfort and support to provide a constant energizing massage as you move. The nodules apply gentle pressure to the nerve endings in the soles of your feet, relieving pain.

3

Wear Them Only One Hour At First

Orthotics work by changing the position of your feet. It may take some time before you are comfortable wearing them all day. To begin, wear the orthotics one hour the first day, two hours the second and so on, until you can wear them comfortably all day long. If you feel discomfort, remove them from your shoes. Begin wearing them the next day without increasing the time they are worn. Then continue to increase usage until they are comfortable. If you feel pain when wearing WalkFit® Platinum orthotics, discontinue use and consult with your physician.

4

Be Patient

It could take as long as two to three weeks to adjust to your WalkFit® Platinum orthotics. If you choose to start out on LOW or MED inserts, after a few weeks you may want to try MED or HIGH to find your ideal fit. Remember, if after a few days the new inserts are not comfortable, go back to MED or LOW for greater comfort. The important thing is to choose the level that feels best to you.

5

Avoid Tight-Fitting Shoes

WalkFit® Platinum orthotics will fit comfortably in most shoes, with the exception of tight-fitting shoes. If you experience this difficulty, try your orthotics with a pair of looser-fitting shoes.

6

Look Ahead To New Foot Comfort

Once your feet have adjusted to the WalkFit® Platinum orthotics, you'll experience a noticeable increase in foot support and comfort. You'll hardly know you're wearing them and you'll always want them in your shoes. We know you will love your WalkFit® Platinum orthotics. Thank you for your purchase.

TO RE-ORDER

visit
www.walkfitplatinum.com



“LOVE IT OR RETURN IT” GUARANTEE

If, within 30 days of receiving WalkFit® Platinum orthotics, you are not thrilled with your results, simply return them for a complete refund of your purchase price (less shipping and processing).

To receive a refund, please call us at 1.877.264.9243 Monday-Friday 7 am - 4 pm PST and Saturday 7 am - 3 pm or email us at customerservice@walkfit.com

RETURN TO: 5360 Capital Ct. Ste. 103 Reno, NV 89502

© 2017-2022 Ideal Living, LLC. All Rights Reserved. U.S. Patent Nos. D509,951; 6,990,756. Chinese Patent No. 200430011865.5. European Registered Design No. 313317-0001. Hong Kong Patent No. 0412508.6. Additional patents pending. Made in China.